

HAICO TELGTE II EQUINE OPERATING TABLE

INSTALLATION GUIDE



Item no. 608750

TELEPHONE +49 7461 96 580 0

www.eickemeyer.com

veterinary technology for life
Eickemeyer®

INSTALLATION GUIDE HAICO TELGTE II EQUINE OPERATING TABLE

Installations of the new table about one hour work as follows:

Installing the Table

Before you start working with the surgery table make the following installation works.


1. Install the fuse to its place (located in the rear end of the table).
Tools: a small screwdriver (3 to 5 mm wide)
2. Install the head part of the table to its place.
Tools: 6 mm allen key, 13 mm spanner, two 17 mm spanners
3. Install the charger on the wall.
Tools: 10 mm drill (type depends on the wall material), cross type screwdriver
4. Install the equipment stand on the wall.
Tools: 10 mm drill (type depends on the wall material), 10 mm spanner, screwdriver (5 to 10 mm wide)
5. Charge the battery before start using the table.

Installing the Fuse




Installing the Head Support

Hasp screws to be installed

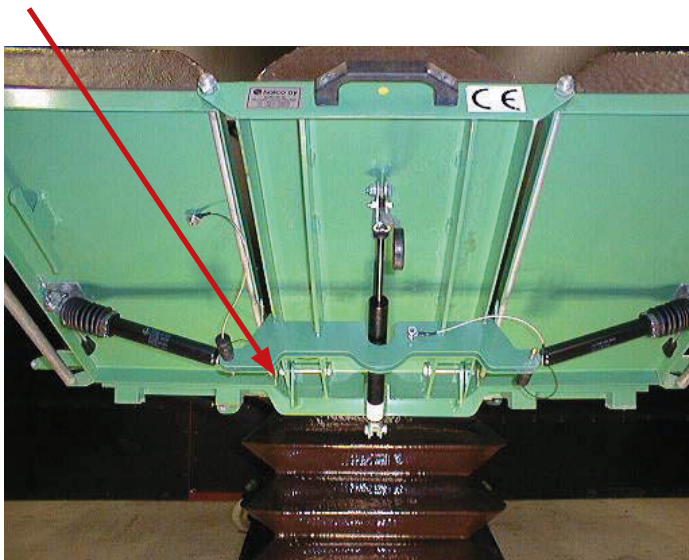

The screws

Screw installation direction

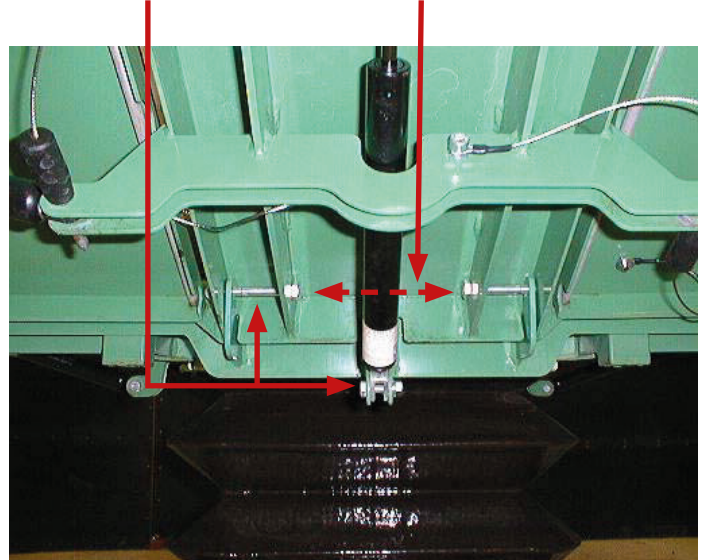

Using the Battery Charger

This automatic charger is designed for 90 to 270 V AC and 50 to 60 Hz.

In order to start the charge the battery terminal voltage must be over 1 V.

In case the terminal voltage is under 1 V the battery may be in short and it should be replaced.

Telgte series tables



Pro3 tables with magnet plug



Note the hanging way



The LEDs here

The built in micro controller controls the charging process.



Yellow LED on

Charge in progress

Max power: **For max power ON – disconnect and connect again the wall plug**

Time: 0 – 12 h



Both LEDs on

Battery almost full

14.4 V charge

Time: 4 h



Green LED on

Charge ready

Maintenance charge 13.7 V

No time limit




Attention!

- Keep the magnetic contacts clean from metal crumbs or connection between the charger and the battery may be cut off.
- As a safety measure, the charger should be mounted on a wall at a height of about 1.5 m.

Equipment Stand

Install the equipment stand on a wall and keep the equipment in order as shown in the photo.

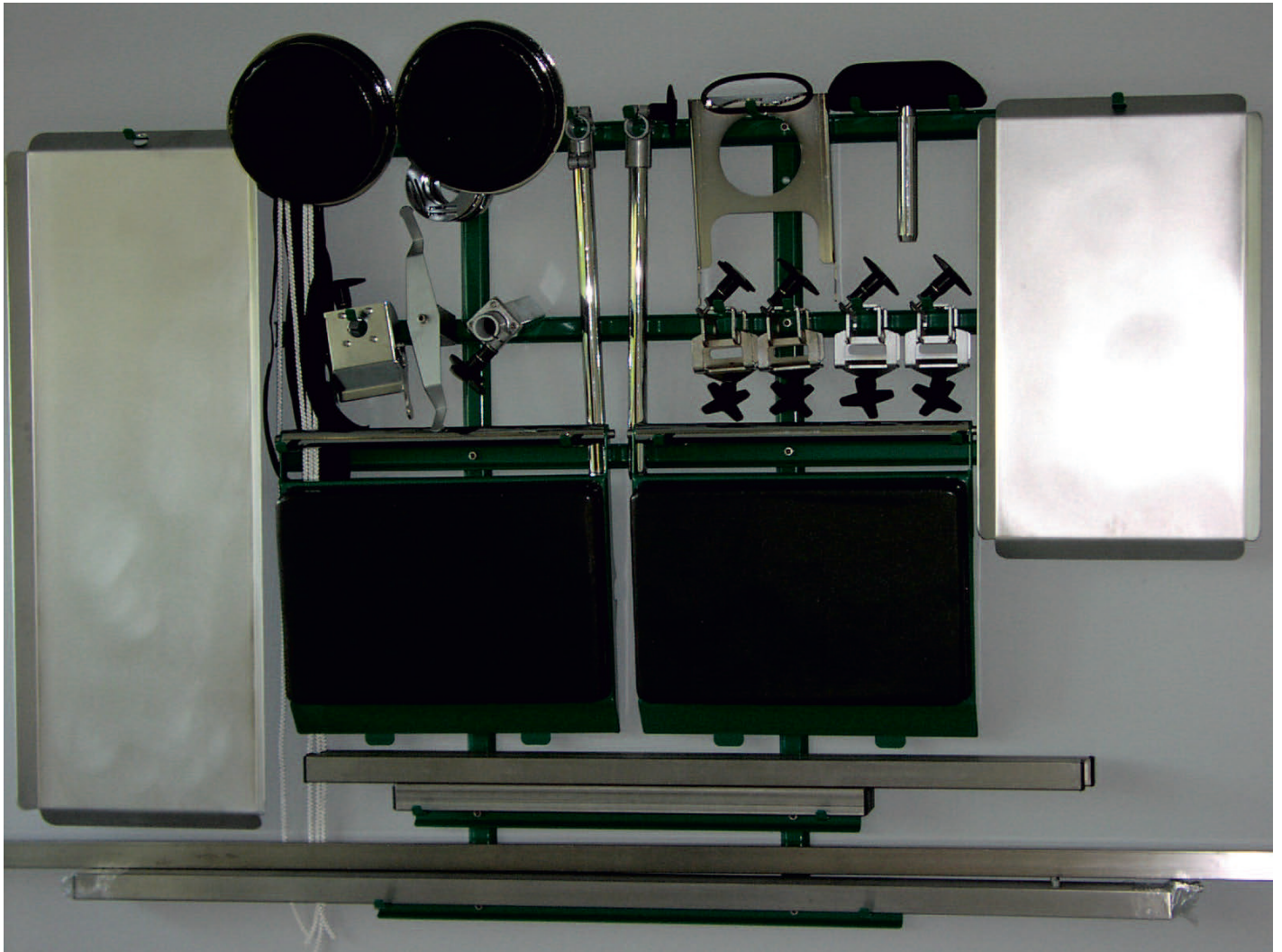

NOTES

GERMANY

EICKEMEYER KG
Eltastraße 8
78532 Tuttlingen
T +49 7461 96 580 0
F +49 7461 96 580 90
info@eickemeyer.de
www.eickemeyer.de

SWITZERLAND

EICKEMEYER AG
Sandgrube 29
9050 Appenzell
T +41 71 788 23 13
F +41 71 788 23 14
info@eickemeyer.ch
www.eickemeyer.ch

POLAND

EICKEMEYER Sp. z o.o.
Al. Jana Pawła II 27
00-867 Warszawa
T +48 22 185 55 76
F +48 22 185 59 40
info@eickemeyer.pl
www.eickemeyer.pl

ITALY

EICKEMEYER S.R.L.
Via G. Verdi 8
65015 Montesilvano (PE)
T +39 085 935 4078
F +39 085 935 9471
info@eickemeyer.it
www.eickemeyer.it

DENMARK

EICKEMEYER ApS
Solbakken 26, Hammelev
6500 Vojens
T +45 7020 5019
info@eickemeyer.dk
www.eickemeyer.dk

NETHERLANDS

EICKEMEYER B.V.
Bellweg 44
4104 BJ Culemborg
T +31 345 58 9400
info@eickemeyer.nl
www.eickemeyer.nl

UNITED KINGDOM

EICKEMEYER Ltd.
3 Windmill Business Village
Brooklands Close
Sunbury-on-Thames
Surrey, TW16 7DY
T +44 20 8891 2007
info@eickemeyer.co.uk
www.eickemeyer.co.uk

CANADA

EICKEMEYER Inc.
617 Douro Street, Suite #205
Stratford, Ont. Canada
N5A 0B5
T +1 519 273 5558
F +1 519 271 7114
info@eickemeyervet.ca
www.eickemeyercanada.ca