Ultrasonic Cleaner

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561008 – 561011

Are there general indications for the use of cleaning concentrates in the ultrasonic cleaner?

A cleaning agent should be sonicated about 5 minutes to degas it. Then it has a better effect.

After that it is to be checked to which temperature the cleaning agent can be heated (**WARNING:** With the lid closed the agent can get hotter than the preset temperature). Then the instruments are to be placed in the agent and left there according to the agent suppliers' advise.

A chart with detailed information of the cleaning time, the dosing of the ultrasound concentrate and the temperature can not be given exactly as this can be different for each user as well as for each use.

For this reason we recommend that certain parameters are defined in advance and all users are moving within these corridors:

For stainless steel parts this would be a temperature setting of max. 60 °C and cleaning intervals of 5–10 minutes. After each interval the items cleaned must be checked to determine whether the cleaning time was sufficient. This must be repeated until the product is completely clean. In the case of materials made of plastic, the temperature must be lowered to 40 °C. The interval method remains the same.

As long as the parts have deposits (dirt) and are therefore cleaned in an ultrasonic bath, also solution can be used to clean the parts completely.

May desinfectant be used in the ultrasonic cleaner?

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